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### GB

#### IMPORTANT

Read this instruction leaflet carefully and keep it for future reference.

The contact lens is intended to be used for one month and needs to be replaced after this period. Discard the used lens after 1 month. Follow the replacement schedule, for the sake of your eyes and to contribute to the enjoyment of your contact lens wear.

Please note the UV-protection afforded by these Contact Lenses complies the protection offered by sunglasses but does not replace it.

HAVE YOUR EYES CHECKED AT LEAST ONCE A YEAR BY YOUR CONTACT LENS OR EYE SPECIALIST.

Like virtually all soft contact lenses, the Unicare contact lenses should not be worn while sleeping. Lenses should be taken out before going to bed.

- Remove the lenses if you experience painful eyes, burning or red eyes, blurred vision, excessive tears, dry eyes, sensitivity to light and ask your doctor or contact lens specialist for advice.
- Do not use unsuitable fluids for your lenses and your contact lens container like tap water, mineral water, saliva and solutions for other types of lenses, but use a special maintenance system, e.g. Unicare.
- Wash your hands thoroughly and regularly clean your lens container.
- Do not shower or swim wearing contact lenses unless you are wearing waterproof swimming goggles.

#### SAFETY MEASURES SOFT CONTACT LENSES

- Usage of medicines can have an adverse effect on wearing contact lenses successfully.
- Keep the lenses and maintenance products away from children.
- Consult your doctor or contact lens specialist in emergencies or if you have any doubts about the information on this instruction leaflet.

#### USE AND MAINTENANCE OF SOFT CONTACT LENSES

- Before you touch the contact lenses with your hands, you should always wash your hands, rinse them thoroughly and dry them with a clean, lint-free towel. This prevents the lens from being contaminated. Never pick up the lenses with any sharp objects, like tweezers. Long or sharp nails can also damage the soft material of the lenses. So please take care.
- Check the package for damage before you remove the contact lenses. If the original packaging is damaged or broken, do not use this contact lens. You may return the damaged lens to the following address: Melleson Pharma, Minervum 7384, NL-4817 ZH Breda, Netherlands


#### INSERTING SOFT CONTACT LENSES

The lenses are packed in a sterile blister package. Peel back the foil and carefully remove the lens from the solution. Place the lens on the tip of your index finger. Check the contact lens has not turned inside out (see picture).

Pull the lower lid down with your middle finger of the same hand, bend forwards, look in the mirror with both eyes, open your eyes as wide as possible and pull the upper lid up with the fingers of the other hand (see picture).


Look down while bending forwards. Gently place the contact lens on the eye and slowly release the eyelids and blink carefully.

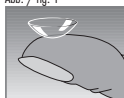Abb. / fig. 1

richtig / correct

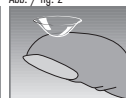Abb. / fig. 2

falsch / incorrect

Abb. / fig. 3

einsetzen / inserting

Abb. / fig. 4

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Repeat these steps for the other lens.

IF YOUR VISION IS BLURRED - check whether you have switched the right with the left lens.

IF IT FEELS VERY UNCOMFORTABLE - remove the contact lens from your eye and check if the lens has not turned inside out (see picture).

IF THE DISCOMFORT CONTINUES - remove the lens and consult your doctor or contact lens specialist.

NEVER PLACE A DEHYDRATED LENS ON THE EYE.

#### REMOVING SOFT CONTACT LENSES

Pull the bottom lid down with your middle finger. Look up and slide the contact lens down to the white of the eye. Carefully pinch the contact lens with the tips of your thumb and index finger and remove it. (see picture)

#### MAINTENANCE OF SOFT CONTACT LENSES

It is vital that you take care of your lenses daily. Unicare solutions are great for this. Always follow the instructions on the bottle; the contact lenses need to be kept in a clean lens container, completely immersed in a fresh solution. If you only wear the contact lenses occasionally, it is advisable to change the preservative solution every 24 hours.

#### TIPS FOR COSMETICS

- Always insert your contact lenses before you apply make-up.
- Remove the lenses before you take off your make-up.
- If your contact lenses do come into contact with cosmetics, clean the lenses with the maintenance solution.
- Keep your eyes closed when using hairspray.
- Use a non-greasy make-up remover.
- Cream eye-shadows prevent powder particles from coming into contact with your contact lenses and causing irritations.
- Never share make-up.
- Never apply eye liner to the inner rim of the eye, only to the outer rim.
- Be careful with aggressive substances like alcohol, acetone, nail varnish and perfume.

#### WARNING

By carefully following the instructions of your specialist, you can prevent problems with your eyes.

Date of issue of these instructions: 12/2014

For help and information contact Melleson Pharma NL at [info@melleson.nl](mailto:info@melleson.nl)

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Contents: sterile, soft contact lenses in a balanced saline solution. Material: Filcon 4, ionic. Water content: 55 %.

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